
AA-17-HEPA

HEPA filter for ADU-17 series



The Aseptico AA-17-HEPA provides HEPA filtration for Aseptico ADU-17 series units.

PACKAGE CONTENTS

<p>HEPA Filter Canister w/ HEPA Filter Insert Installed</p>	 A clear plastic HEPA filter canister with a grey HEPA filter insert installed inside. Two white flexible hoses are attached to the top of the canister, one with a blue connector and the other with a white connector.
<p>Carrying Case</p>	 A black plastic carrying case with a black handle and a latch on the lid.
<p>Spare HEPA Filter Insert (P/N 730816)</p>	 A cylindrical HEPA filter insert with a grey mesh outer layer and a white pleated inner filter.
<p>Instruction Manual</p>	 The instruction manual cover for the AA-17-HEPA filter. It features the title "AA-17-HEPA" and "HEPA filter for ADU-17" at the top. Below the title is an image of the carrying case and the HEPA filter canister. At the bottom, it says "INSTRUCTION MANUAL" and the Aseptico logo.

OVERVIEW



SETUP

1. Remove the waste tank **A** and water bottle **B** from the ADU-17.



2. Connect the vacuum hose from the top of the AA-17-HEPA canister to the vacuum hose fitting **A** on the ADU-17. Connect the vacuum hose from the side of the AA-17-HEPA canister to the white fitting **B** on top of the waste tank.



3. Install the waste tank **A** and water bottle **B** back into the ADU-17. Reconnect the HVE and saliva ejector lines to the the waste tank as described in your ADU-17 instruction manual.



4. The AA-17-HEPA is shipped with one spare HEPA filter insert in the tray of the carrying case. Replace the HEPA filter insert annually or sooner if you notice reduced vacuum performance of the ADU-17. To access the filter insert, open the 3 clips **A** on the base of the AA-17-HEPA canister. Dispose of the used HEPA filter **B** in compliance with regional regulations. Contact Aseptico at the info provided on back cover to order replacement HEPA filter inserts (P/N 730816).





P.O. Box 1548, Woodinville, WA 98072
8333 216th Street S.E., Woodinville, WA 98072
Phone (425) 487-3157 • Toll Free (800) 426-5913
Web: www.aseptico.com
Email: info@aseptico.com