


# ADC-02CF • Setup Guide


Portable Hydraulic Dental Chair • NSN: 6520-01-652-8993

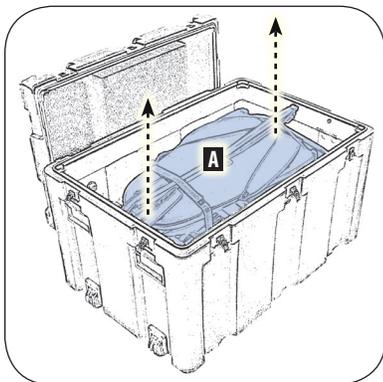


1. Using two people, lift the chair **A** from the transport case and set it on the floor.



2. Pull out the stabilizer **A** and use it as a handle to tilt the chair up on its wheels **B**, then roll the chair to where it will be used.

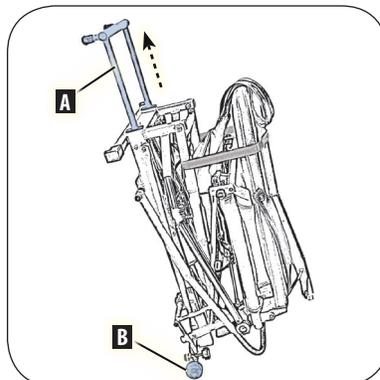


3. Set the chair back down on the floor making sure the stabilizer **A** is fully extended. Remove the backrest strap **B** from the chair.

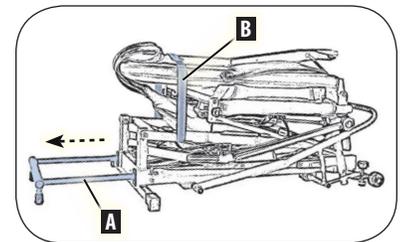


CONTINUED ON BACK...


4. Adjust the leveling foot **A** as needed to compensate for an uneven floor.

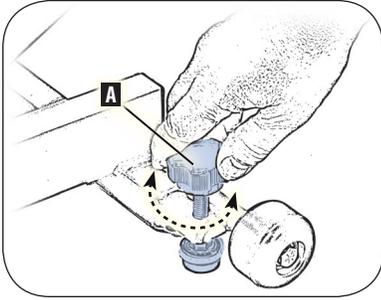


5. Rotate the selector lever **A** to the SEAT LIFT position as shown on the label **B** on the chair base.

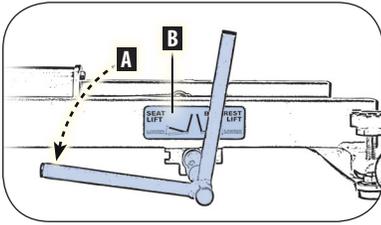




Pump the lift lever **C** to raise the chair to the desired height.

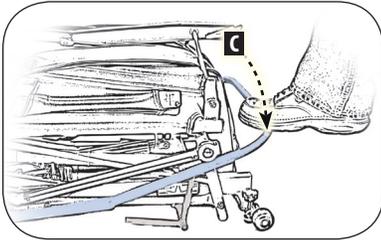


Press and hold down the selector lever **D** in the SEAT LOWER position to lower the chair.

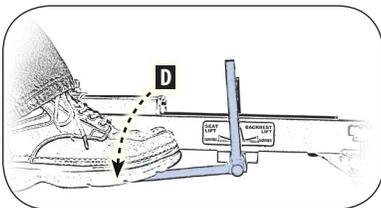


6. Lift the backrest **A** upright while guiding the clevis pin **B** on the end of the backrest cylinder, into the latch **C** on the backrest frame, until they have securely latched **D**.

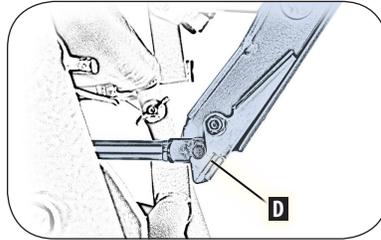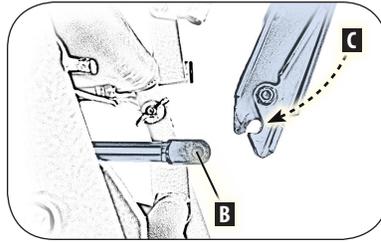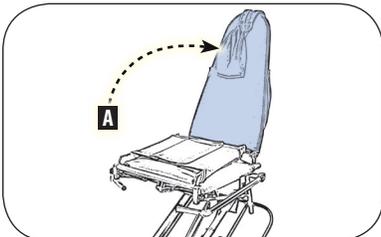

7. Rotate the selector lever **A** to the BACKREST LIFT position as shown on the label **B** on the chair base.

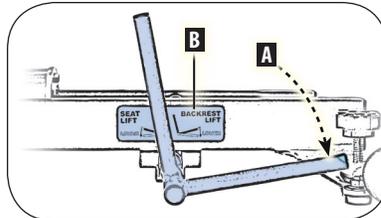


Pump the lift lever **C** to raise the backrest to the desired height.

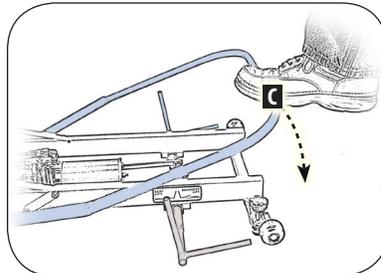


**i** Never attempt to raise the backrest by lifting up on the backrest, as this may pull air into the hydraulic system! Always use the lift pump lever to raise the backrest.



Press and hold down the selector lever **D** in the BACKREST LOWER position to lower the backrest; if the seat is unoccupied, it may be necessary to push down on the backrest to help it recline.

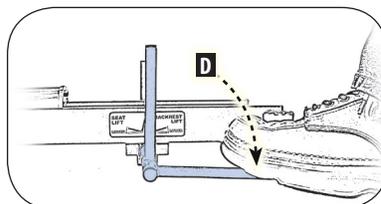

8. Unfold the leg **A** and foot **B** panels of the chair.

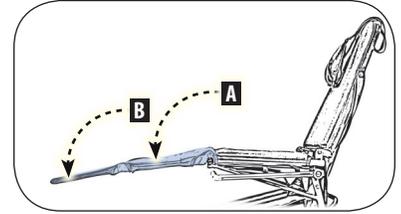

9. Fold out the hand grips **A** on the arm posts and insert them into the ends of the arm slings **B**.

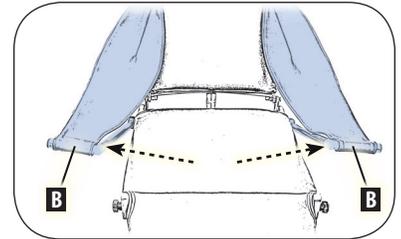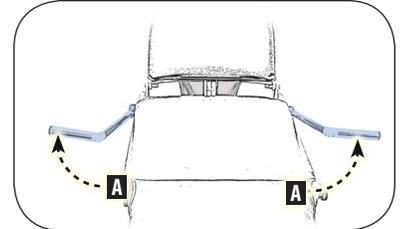

10. Adjust the height of the pillow **A** as needed.



**i** Refer to the Operation and Maintenance Manual for complete instructions.

**Aseptic**

P.O. Box 1548 • Woodinville, WA 98072-1548  
(800) 426-5913 • (425) 487-3157  
www.aseptic.com • info@aseptic.com

P/N: 421061 • Rev. C • ECO 13890 • 8/2016  
Printed in USA