

HANDIDAM®-LF INSTRUCTION MANUAL

The Aseptico **HandiDam®-LF** dental dam saves time, aids in infection control, and preserves patient goodwill. The latex-free dam is manufactured exclusively with synthetic rubber to avoid patient exposure to latex allergens.

Package Contents:

| <u>Item No.</u> | <u>Dams Qty.</u> | <u>Sticks Qty.</u> |
|-----------------|------------------|--------------------|
| I-70LF | 20 dams | 5 sticks |
| I-70LF-100 | 100 dams | 20 sticks |

Instructions:

- 1) Optional: Insert one of the plastic sticks into the two holes in the HandiDam®-LF frame as shown in the picture. This serves as a handle when taking radiographs, and comfortably contours the dam to the patient's face.
- 2) Use the dam horizontally as pictured. Punch a hole and use the appropriate clamp, as you would in any standard rubber dam technique.
- 3) When taking radiographs, just fold the dam out of the way to insert film.
- 4) When procedure is complete, discard the HandiDam®-LF and frame. Autoclave sticks according to sterilization® procedure below and save for re-use.

Sterilization Procedure:

Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.



**AVOID CROSS-CONTAMINATION
WITH LATEX BASED PRODUCTS**



The Aseptico HandiDam is a single-use product. Discard after one use.




Protect from light
Store in cool environment
(Room temperature or cooler)



Manufacturer



Aseptico Inc
8333 216 St SE
Woodinville
WA 98072
USA

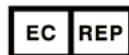

Advena Ltd
Tower Business Centre, 2nd Flr,
Tower Street, Swatar, BKR 4013
Malta



LATEX FREE

HandiDam[®]-LF
Pre-framed
Synthetic Rubber Dam



HandiDam[®]-LF

Pre-framed Synthetic Rubber Dam

**LATEX FREE, EASY TO USE,
CONVENIENT, AND SAVES TIME**



P.O. Box 1548 • Woodinville, Washington 98072-1548
1-800-426-5913 • 425-487-3157 • Fax: 425-487-2608
www.aseptico.com • info@aseptico.com