








**LATEX!**  
This product contains latex rubber, which may cause allergic reactions.

 The Aseptico HandiDam is a single-use product. Discard after one use.

 Protect from light  
Store in cool environment (Room temperature or cooler)

 Manufacturer

 Aseptico Inc  
8333 216 St SE  
Woodinville  
WA 98072  
USA

 Advena Ltd  
Tower Business Centre, 2nd Flr,  
Tower Street, Swatar, BKR 4013  
Malta

HandiDam<sup>®</sup> is manufactured exclusively by Aseptico. It saves time, aids in infection control, and preserves patient goodwill.

# HandiDam<sup>®</sup>

## Pre-framed Rubber Dam



Easy to use, convenient,  
and saves time.


# Instructions for Use



**1.** Insert the stick through the holes as pictured.  
(Use of stick is optional)



**2.** The dam should be curved inward as shown

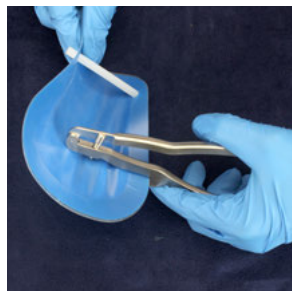


**3.** Punch the hole for your dam clamp.



**4.** Place your clamp through the hole as shown



**5.** Flip the loose side of the dam back for better access.

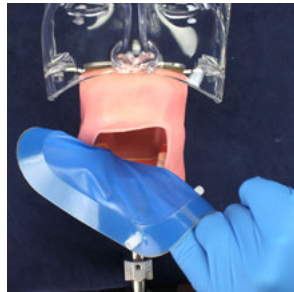

**6.** Pull on the stick for easy access to mouth for suctioning or inserting radiograph film.

## Autoclave sticks and save for re-use.

**Sterilization Procedure:** Sterilize the plastic sticks between each patient use.  
Steam autoclave at 132° C (275° F) for 3 minutes.

## Package Contents:

<u>Item No.</u>	<u>Dams Qty.</u>	<u>Sticks Qty.</u>
I-70BM	20 dams	5 sticks
I-70BM-100	100 dams	20 sticks