

# OPERATION AND MAINTENANCE MANUAL

## Portable Pediatric Dental Chair

ADC-01P (gray)  
ADC-01P-RED (red)  
ADC-01P-BLU (blue)



**Aseptico**

## **TABLE OF CONTENTS**

<b>Introduction</b> .....	3
Purpose .....	3
Performance Characteristics .....	3
Items Furnished .....	3
<b>Operation Instructions</b> .....	4
Chair Set-Up .....	4
Chair Adjustments .....	5
Collapsing the Chair .....	6
<b>Cleaning &amp; Maintenance</b> .....	6
<b>Parts List</b> .....	8
<b>Parts Diagram</b> .....	9
<b>Warranty</b> .....	Back Cover

## **TABLE OF ILLUSTRATIONS**

<b>Figure 1 - Leg Adjustment Pin Location</b> .....	4
<b>Figure 2 - Leg Adjustment Pin Placement</b> .....	4
<b>Figure 3 - Folding Out Foot/Leg Rest</b> .....	4
<b>Figure 4 - Arm Sling Pin Insertion</b> .....	4
<b>Figure 5 - Backrest adjustment</b> .....	5
<b>Figure 6 - Positioning footrest to vertical</b> .....	5
<b>Figure 7 - Parts Diagram</b> .....	8

## INTRODUCTION:

Aseptico's Portable Pediatric Dental Chair is intended to provide adolescent patient seating for field dentistry and medical treatment. The chair is fully adjustable from upright to supine positions and is constructed from a high strength steel frame and durable replaceable coverings.

Available in three colors:

- Gray (ADC-01P)
- Red (ADC-01P-RED)
- Blue (ADC-01P-BLU)

## PERFORMANCE CHARACTERISTICS:

### Dimensions

Chair- Folded: 37" W x 23" L x 9.5" H  
(94 cm W x 58 cm L x 24 cm H)  
Extended: 67" W x 28" L x 46" H  
(170 cm W x 71 cm L x 117 cm H)  
Soft carrying case: 36" W x 24" L x 10" H  
(91 cm W x 61 cm L x 25 cm H)

### Weight

Chair only: 30 lbs.  
In soft carrying case: 34.5 lbs.  
Maximum Load Capacity: Approx. 300 lbs.

### Range of Positions

Minimum Seat Height: 11"  
Maximum Seat Height: 22"  
Backrest: Adjustable from upright (vertical) to supine (horizontal) positions  
Footrest: Adjustable from upright (vertical) to supine (horizontal) positions

### Expected Service Life:

10 years

### Essential Performance:

This device does not have ESSENTIAL PERFORMANCE as defined in IEC 60601-1.

## ITEMS FURNISHED

Portable Pediatric Dental Chair (ADC-01P, ADC-01P-RED or ADC-01P-BLU)  
Soft padded carrying case with handles  
Operation and Maintenance Manual

## SAFETY PRECAUTIONS:

*Aseptico accepts no liability for direct or consequential injury or damage resulting from improper use, arising in particular through the non-observance of the operating instructions, or improper preparation and maintenance of this product.*



Federal (United States) law restricts this device to sale on the order of the dentist.

To prevent injury to people and damage to property, please heed relevant warnings and remarks. They are marked as follows:

- **WARNING:** Serious injury or death may result if ignored.
- **CAUTION:** Damage to property or the environment may result if ignored.
- **NOTE:** Important additional information and hints.

**WARNING:** Use for intended purposes only. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair. Before using this product, make sure that you have studied and understood the operating instructions.

**WARNING:** Do not sit on the leg panel or foot panel. Patients must sit on the main seat to reduce the risk of a tipping hazard.



**WARNING:** Always use the chair on a hard flat surface.

**WARNING:** The chair must be unoccupied when making height adjustments to the chair.

**CAUTION:** Before using the Chair, ensure that the leg adjustment pins have been fully inserted through the adjustment holes in the legs.

**CAUTION:** Do not alter the chair assembly.

**CAUTION:** All repairs are to be performed by authorized service personnel only.

**NOTE:** Report any malfunction or deterioration in the characteristics and/or performance of this device, as well as any inadequacy in its labelling or the instructions for use which might lead to or might have led to the death of a patient or user or to a serious deterioration in their state of health (e.g. Serious Adverse Event) immediately to the manufacturer and the regulatory authority where the incident occurred.

## CHAIR SET-UP:

1. Remove the chair from its carrying case and set it on the floor, legs down, in its folded position.
2. Rotate the chair up onto one side and remove the leg adjustment pins from their storage pouch underneath the chair seat. (Fig. 1)

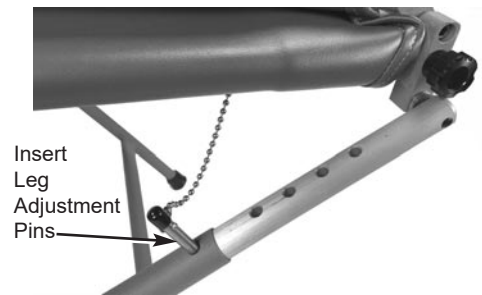


Figure 2 - Leg Adjustment Pin Placement

4. When the chair has been raised and the height set, the foot and leg portion should be folded forward, out horizontally. It will stop in the horizontal position. (Fig. 3)

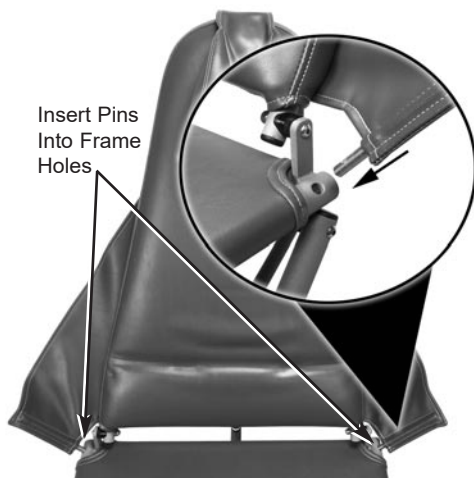


Figure 4 - Arm Sling Pin Insertion

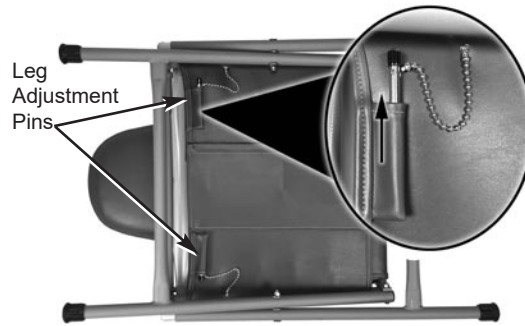


Figure 1 - Leg Adjustment Pin Location

3. Standing in front of the chair, put your toe on one of the front chair legs and lift up on the backrest and front seat portion. Raise chair seat to desired height. Lock in place by inserting leg adjustment pins through chair leg tube and into adjustment link on the right and left side of the chair. (Fig. 2)



Figure 3 - Folding Out Foot/Leg Rest

5. Insert right and left arm sling pins into holes on the sides of the seat frame. (Fig. 4)
6. Complete set-up by positioning backrest and placing headrest at the proper height on the front side of backrest.

## CHAIR ADJUSTMENTS:

1. **Seat Height-** The chair seat height can be set at any of the five preset settings. Without a patient in the chair, put your toe on one of the front chair legs and lift up slightly on the chair seat. While lifting and holding seat, remove the leg adjustment pin from the right and left side. Align link with tube hole in any of the five positions and lock in place by inserting the pin and chain assembly. (see Fig. 2)
2. **Backrest-** The backrest may be set in any position from upright to full supine. To change the position of the backrest, simply lift the chrome friction lever on the back side of the backrest, lower backrest, then release lever to lock. (see Fig. 5)

### WARNING

NEVER LOWER THE BACKREST WHEN A PATIENT IS IN THE CHAIR WITHOUT COMPLETELY SUPPORTING THEIR WEIGHT FIRST.

To return the backrest to an upright position, lift backrest to desired position. Backrest will automatically lock in place. (Fig. 5)

3. **Footrest-** The footrest may be positioned in the normal horizontal operating position, or allowed to hang down vertically so a patient can be seated in a sitting position. To release footrest into the vertical position, first fold toe section inward, then lift slightly on footrest section. While lifting, pull the black knobs on each side of the chair frame to release and push footrest down into the vertical position. Lock in the vertical position by snapping holding strap around the chair leg frame. To reposition footrest in the horizontal position, unsnap holding strap from frame, lift footrest to the horizontal position until it locks, then unfold toe section out to horizontal. (Fig. 6)

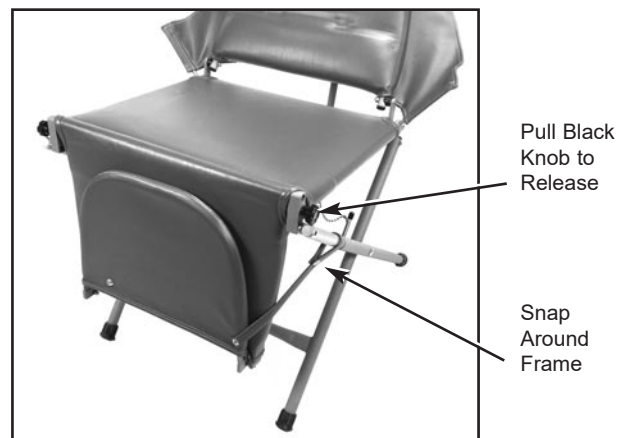


Figure 6 - Positioning footrest to vertical

4. **Headrest-** The headrest slides freely on its mounting strap and can be positioned at different heights for different size patients.



Figure 5 - Backrest adjustment

## **COLLAPSING THE CHAIR:**

### **WARNING**

NEVER FORCE THE CHAIR INTO THE COLLAPSED POSITION. FORCING THE CHAIR DOWN WITHOUT RELEASING THE BACKREST ADJUSTMENT LEVER MAY BEND THE ADJUSTMENT ASSEMBLY AND WILL VOID THE WARRANTY.

1. Fold toe and then leg sections onto seat as it was originally stored.
2. Pull arm slings out of the holes in the chair frame.
3. Put your toe onto one of the front chair legs and lift up slightly on the seat, then remove height adjustment pins. While supporting chair weight, let the chair fold down toward the floor.
4. As the chair folds down, release the backrest adjustment lever until the chair collapses completely. Rotate the chair up onto one side and insert the leg adjustment pins into their storage pouch underneath the chair seat.
5. Fold the arm slings and headrest onto the chair and place folded chair into carrying case.

## **CLEANING AND MAINTENANCE:**


Although bleach, when diluted to <10% and wiped clean immediately could be used, it is not recommended. The use of barrier covers and mild cleaning solutions is recommended as the primary asepsis approach on chair and stool upholstery instead of relying on chemicals. The use of barriers greatly extends the life of chair and stool upholstery and can be replaced between patients.



## PARTS LIST

	Item	ADC-01P	ADC-01P-RED	ADC-01P-BLU	Quantity
1	Seat Cover Assembly	460451	460451-01	462551	1
2	Armrest Right	460447	460447-01	462550	1
3	Armrest Left	460447	460447-01	462550	1
4	Cover, backrest	462710	461845	462711	1
5	Strap, headrest	460449	461847	462538	1
6	Screw, #8x5/8" Chrome SEMS	510239	<—	<—	8
7	Headrest, pillow assembly	462712	461846	461713	1
8	Armrest support rod assembly	460444-08	<—	<—	2
9	Tie Wrap, 30"	510135	<—	<—	6
10	Tie Wrap, 13"	510134	<—	<—	3
11	Strap, leg	460448	461848	462539	1
12	C/S, 1/4-20x1" BtnSoc S/S	510294	<—	<—	1
13	Hinge, back support	460382-08	<—	<—	1
14	Nut, 1/4-20 Nyloc S/S	510296	<—	<—	8
15	Back Support Assembly	330088	<—	<—	1
16	Leg adjustment link	460097-08	<—	<—	2
17	Hole plug, 1/4" black	510201	<—	<—	2
18	Endcap, 1" domed	850026	<—	<—	4
19	Leg Assembly, inside	330099	<—	<—	1
20	Leg Assembly, outside	330101	<—	<—	1
21	Bolt, 5/16-18 Grade 5 carriage	510242	<—	<—	2
22	Washer, 5/16 Flat S/S	510266	<—	<—	4
23	Nut, 5/16-18 Nyloc S/S	510126	<—	<—	2
24	Endcap, 1" insert	510204	<—	<—	2
25	C/S, 1/4-20x1-1/4" BtnSoc S/S	510295	<—	<—	3
26	Toe Frame Assembly	330091	<—	<—	1
27	Nut, 5/16-18 hex S/S	510103	<—	<—	2
28	Foam, seat pad	460576	<—	<—	1
29	Spring Pin, 1/4x3/4" S/S	510251	<—	<—	2
30	Leg Frame Assembly, cml	330093-08	<—	<—	1
31	Knob, 5/16-18 insert cornet	850025	<—	<—	2
32	Spring, .420Dx.038Wx.875L comp.	510233	<—	<—	2
33	C/S, 5/16-18x1-1/4" S/S	510050	<—	<—	2
34	C/S, 5/16-18x1" S/S	510048	<—	<—	4
35	SMS, PanPhl #6 x 5/8" Tapping	510362	<—	<—	2
36	Washer, 5/16 SAE S/S	510107	<—	<—	8
37	Seat Frame Assembly, complete	330097	<—	<—	1
38	Pin, leg adjustment assembly	460445-08	<—	<—	2
39	C/S, 1/4-20x1-3/4" BtnSoc S/S	510027	<—	<—	4
40	Hole plug, 3/4" black	510263	<—	<—	2
41	Backrest Frame Assembly	330561-01	<—	<—	1
42	Spacer, 1/4ID x 3/4OD Coved	510261-01	<—	<—	2
43	Spacer, 1/4ID x 3/4OD x .250 thick	510379	<—	<—	2
Not Shown	Case, grey carrying, w/Handles	410193	<—	<—	1
Not Shown	Backrest Assembly, complete	330815	330562	330816	1



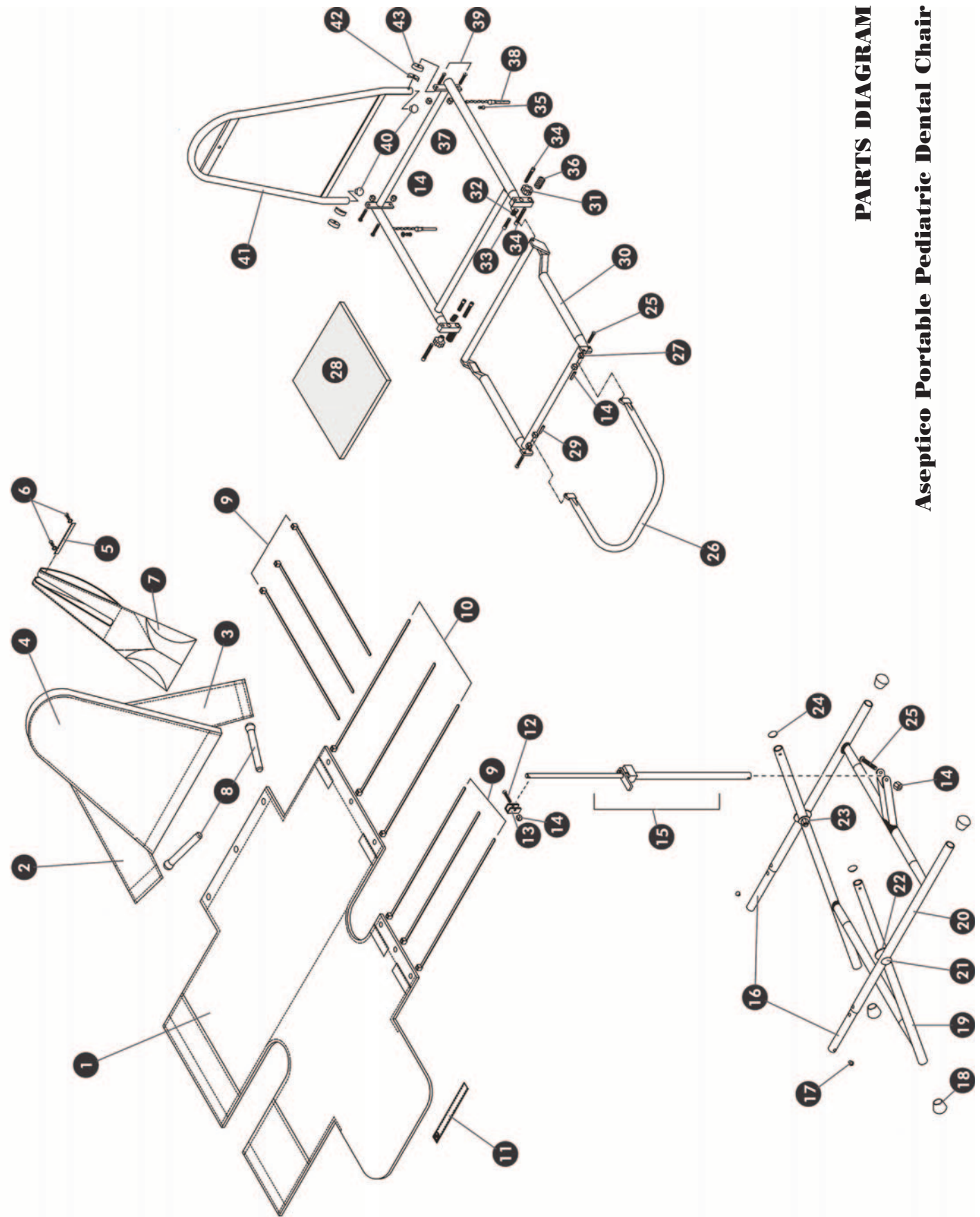

**PARTS DIAGRAM**  
**Aseptico Portable Pediatric Dental Chair**

Figure 7 - Parts Diagram

## SYMBOL DEFINITIONS:



Consult Instructions for Use

---



Manufacturer

---



Part Number

---



Serial Number

---




UDI: Unique Device Identifier

---



MD - Medical Device

---

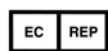

Authorized European  
Representative

---



Caution

---



Country of Manufacture - USA

---



Dental Use Only

Rx Dental Use Only

---



Do not sit on leg panel or foot  
panel.

---



Authorized United Kingdom  
Representative



## WARRANTY

Aseptico Inc. warrants its new products against defects in material and workmanship under normal and proper use, care, and maintenance for a period of two (2) years from date of original invoice. This two (2) year warranty does NOT apply nor is it extended to products that are not manufactured by Aseptico. These products may be covered by a separate limited warranty provided by the particular manufacturer, and all claims and questions regarding the same are to be directed to the particular manufacturer.

Expendable components, such as batteries, fuses, light bulbs, and tubing sets installed on Aseptico products are specifically excluded and have no warranty. Consumable goods are warranted for the stated expiration date of such goods.

Repair or replacement of any product(s) or part(s) under this warranty does not extend the term of this warranty, and such product(s) or part(s) shall remain covered by the unexpired portion of the warranty period, or for ninety (90) days from the date of return to Aseptico, whichever is later. This limited warranty applies only to the initial or first installation of the product or part.

During the specific warranty periods set forth above, Aseptico will, at its option, repair or replace the product(s) or particular part(s) that are found to be defective in either material or workmanship in part or whole. Aseptico shall be the sole arbiter of such action. In the event of alleged defect under warranty, the purchaser is to notify Aseptico's Customer Service department promptly. Customer Service will provide Return Material Authorization (RMA) instructions, usually directing that the product be returned for service, shipping prepaid by the buyer or end user, to Aseptico or its designated and authorized warranty service center.

This warranty shall not apply to products (1) that have been subjected to neglect, abuse, misuse, improper installation, inadequate maintenance, or damage due to improper use of cleaning materials or chemicals, or non-compliance with Aseptico's storage, installation, operation, maintenance or environmental requirements; (2) that have undergone any modification or repair not previously authorized by Aseptico in writing, or service, repair or modification by or from any facility other than an authorized Aseptico service center or technician, or that use non-authorized software or spare or replacement parts; or (3) that fail due to reasonable and normal use or wear and tear, or materials made, furnished or specified by the buyer or end user.

Aseptico does not assume under this warranty any risks or liabilities arising from the clinical use of its products, whether or not such use involves coincidental utilization of products manufactured by others. Under no circumstances will Aseptico be liable or responsible for special, compensatory, incidental, consequential or punitive damages, lost profits, lost sales, or loss of use or loss of business opportunity by or through the use of the product. Aseptico's sole and maximum liability with respect to the product, other than its obligations set forth above, shall be the total purchase price paid for the product.



8333 216th Street SE • Woodinville, WA 98072-1548  
425-487-3157  
[www.aseptico.com](http://www.aseptico.com)



Advena Ltd,  
Tower Business Centre, 2nd Flr,  
Tower Street, Swatar,  
BKR 4013  
Malta



Advena Ltd UK  
Pure Offices  
Plato Close  
Warwick CV34 6WE  
United Kingdom