

---

---

# ADC-08CF

## Portable Dental Stools

NSN: 6520-01-456-5406

---

---

### COMMAND•AIR



**OPERATION / MAINTENANCE  
MANUAL & PARTS LIST**



## **TABLE OF CONTENTS**

<b>Introduction</b> .....	3
Purpose .....	3
Performance Characteristics .....	3
Items Furnished .....	3
<b>Operation Instructions</b> .....	4
Stool Set-Up .....	4
Collapsing the Stool .....	4
<b>Cleaning &amp; Maintenance</b> .....	4
<b>Parts Diagram</b> .....	6
<b>Parts List</b> .....	7
<b>Warranty</b> .....	Back Cover

## **TABLE OF ILLUSTRATIONS**

<b>Figure 1 - Back Support Position</b> .....	4
<b>Figure 2 - Fitting Cross Member Into Stop Channel</b> .....	4
<b>Figure 3 - Leg Height Adjustment</b> .....	4
<b>Figure 4 - Parts Diagram</b> .....	6

## **PURPOSE:**

Aseptico ADC-08CF Command Air Portable Operator Stools provide seating for a doctor and an assistant during field dentistry and medical treatment. The padded seats and backrests make these stools comfortable for extended use, while the folding design and light weight ensure maximum portability. The ADC-08CF Stools are built with high strength steel frames and rugged vinyl coverings, allowing for long term deployment.

## **PERFORMANCE CHARACTERISTICS:**

### **Dimensions**

Stool- Folded: 19" x 20" x 8.5"

Extended: 22" x 20" x (Max Height: 32" Min Height: 29")

Soft carrying case: 19" x 21" x 9"

Mil. spec case: 25.5" x 24" x 19"

Mil. spec case volume: 6.73 Cu. Ft.

### **Weight**

One (1) stool only: 14 lbs.

One (1) stool in soft carrying case: 16.5 lbs.

Two (2) stools & soft cases in mil. spec. case: 60 lbs.

Maximum Load Capacity: Approx. 300 lbs.

### **Range of Positions**

Minimum Seat Height: 19"

Maximum Seat Height: 22"

## **ITEMS FURNISHED**

Two (2) ADC-08CF Command Air Portable Dental Stools

Two (2) Soft padded carrying case with handles

One (1) Mil-spec hard shell case

One (1) Operation and Maintenance Manual

## STOOL SET-UP:

1. Remove the stool from its carrying case and swivel the back support to the upright position. (Fig. 1)
2. Turn the ADC-08CF Stool upside down and set on the edge of an elevated flat surface such as a table or counter top. Scissor the leg frame so that the frame cross member lines up with the stop channel at the rear of the stool on the underside. Fit the frame cross member into the channel and push firmly into place (Fig. 2). Wrap restraining strap over the top of the cross member and snap into place. Ensure that the strap retains the frame cross member firmly into the stop channel.
3. Extend the legs to the proper height by pulling the adjustable legs out of the frame. The adjustable legs are marked with a red height indicating dot. Pull the leg out until the red dot is just exposed and rotate until red dot is in line with the holes in the frame tube. The spring loaded adjustment pin should then pop through the frame hole and lock leg into place. (Fig. 3) Repeat for all other legs.
4. Once you have set the legs to the proper height, turn the ADC-08CF Stool upright and set on a hard, flat surface for use.

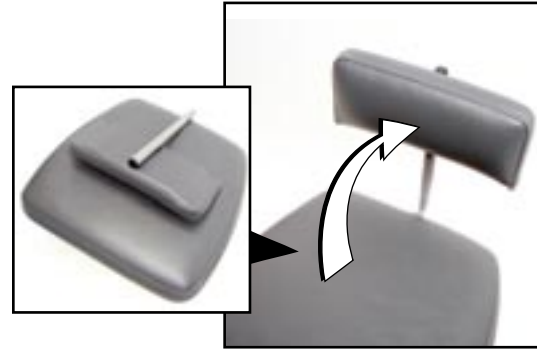


Figure 1 - Backrest Positioning

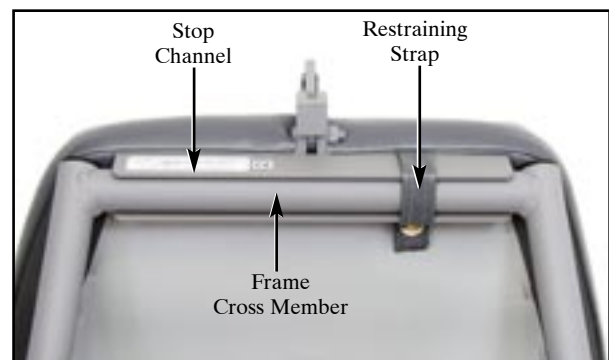


Figure 2 - Securing Leg Frame

**WARNING**

**MAKE SURE ALL LEGS ARE EXTENDED TO THE SAME HEIGHT.**

**NEVER SIT ON THE EDGE OF THE SITTING SURFACE. ALWAYS SIT IN THE CENTER OF THE SEAT.**

## COLLAPSING THE STOOL:

Reverse the set-up procedure to collapse stool for packing. When collapsing the stool, depress spring loaded adjustment pin and turn adjustable leg 90 degrees before pushing leg into frame. This will prevent the legs from stopping at every adjustment interval.

## CLEANING AND MAINTENANCE:

Although bleach, when diluted to <10% and wiped clean immediately could be used, it is not recommended. The use of barrier covers and mild cleaning solutions is recommended as the primary asepsis approach on chair and stool upholstery instead of relying on chemicals. The use of barriers greatly extends the life of chair and stool upholstery and can be replaced between patients.

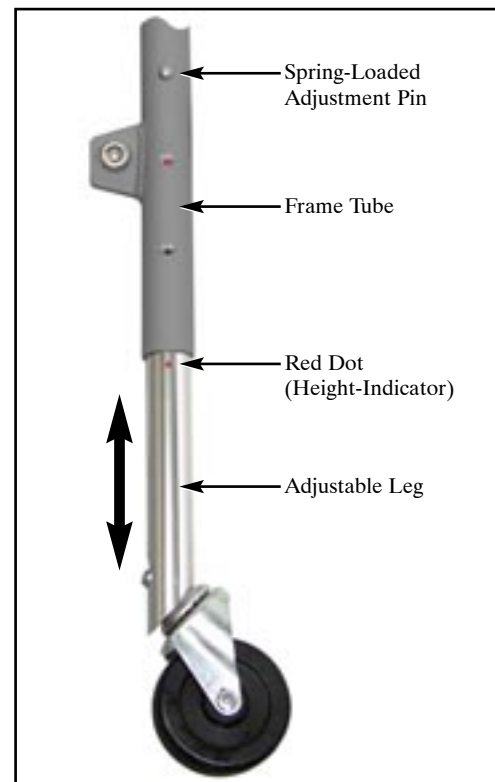


Figure 3 - Leg Height Adjustment

















## **WARRANTY:**

Aseptico Inc. warrants its new products against defects in material and workmanship under normal and proper use, care, and maintenance for a period of two (2) years from date of original invoice. This two (2) year warranty does NOT apply nor is it extended to products that are not manufactured by Aseptico. These products may be covered by a separate limited warranty provided by the particular manufacturer, and all claims and questions regarding the same are to be directed to the particular manufacturer.

Expendable components, such as batteries, fuses, light bulbs, and tubing sets installed on Aseptico products are specifically excluded and have no warranty. Consumable goods are warranted for the stated expiration date of such goods.

Repair or replacement of any product(s) or part(s) under this warranty does not extend the term of this warranty, and such product(s) or part(s) shall remain covered by the unexpired portion of the warranty period, or for ninety (90) days from the date of return to Aseptico, whichever is later. This limited warranty applies only to the initial or first installation of the product or part.

During the specific warranty periods set forth above, Aseptico will, at its option, repair or replace the product(s) or particular part(s) that are found to be defective in either material or workmanship in part or whole. Aseptico shall be the sole arbiter of such action. In the event of alleged defect under warranty, the purchaser is to notify Aseptico's Customer Service department promptly. Customer Service will provide Return Material Authorization (RMA) instructions, usually directing that the product be returned for service, shipping prepaid by the buyer or end user, to Aseptico or its designated and authorized warranty service center.

This warranty shall not apply to products (1) that have been subjected to neglect, abuse, misuse, improper installation, inadequate maintenance, or damage due to improper use of cleaning materials or chemicals, or non-compliance with Aseptico's storage, installation, operation, maintenance or environmental requirements; (2) that have undergone any modification or repair not previously authorized by Aseptico in writing, or service, repair or modification by or from any facility other than an authorized Aseptico service center or technician, or that use non-authorized software or spare or replacement parts; or (3) that fail due to reasonable and normal use or wear and tear, or materials made, furnished or specified by the buyer or end user.

Aseptico does not assume under this warranty any risks or liabilities arising from the clinical use of its products, whether or not such use involves coincidental utilization of products manufactured by others. Under no circumstances will Aseptico be liable or responsible for special, compensatory, incidental, consequential or punitive damages, lost profits, lost sales, or loss of use or loss of business opportunity by or through the use of the product. Aseptico's sole and maximum liability with respect to the product, other than its obligations set forth above, shall be the total purchase price paid for the product.



P/N 420173  
Rev. J  
ECO 15499  
04/2022

*P.O. Box 1548 Woodinville, WA 98072-1548  
1-800-426-5913 \* 425-487-3157 \* Fax: 360-668-8722  
email: [info@aseptico.com](mailto:info@aseptico.com) \* Internet: [www.aseptico.com](http://www.aseptico.com)*