




# Instructions for Use

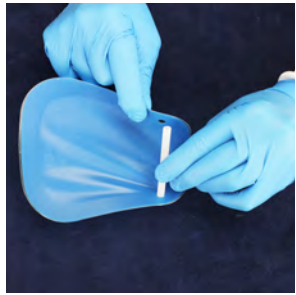

**1.** Insert the stick through the holes as pictured. (Use of stick is optional)

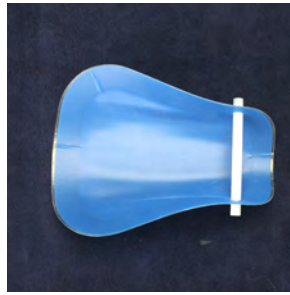

**2.** The dam should be curved inward as shown

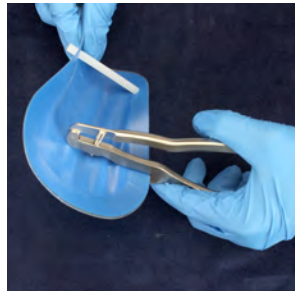


**3.** Punch the hole for your dam clamp.



**4.** Place your clamp through the hole as shown



**5.** Flip the loose side of the dam back for better access.

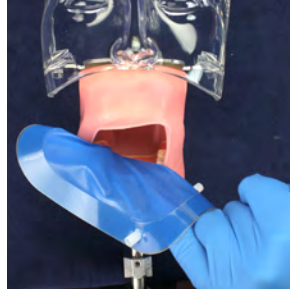

**6.** Pull on the stick for easy access to mouth for suctioning or inserting radiograph film.

**Autoclave sticks and save for re-use.**

**Sterilization Procedure:** Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.


## Package Contents:

Item No.	Dams Qty.	Sticks Qty.
I-70BM	20 dams	5 sticks
I-70BM-100	100 dams	20 sticks


# Instructions for Use



**1.** Insert the stick through the holes as pictured. (Use of stick is optional)

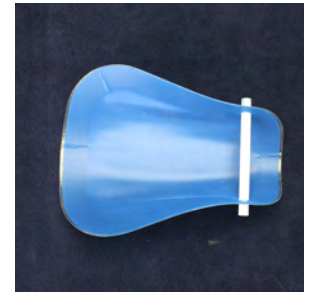

**2.** The dam should be curved inward as shown

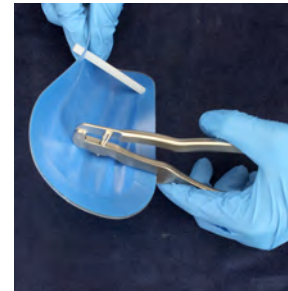


**3.** Punch the hole for your dam clamp.



**4.** Place your clamp through the hole as shown



**5.** Flip the loose side of the dam back for better access.

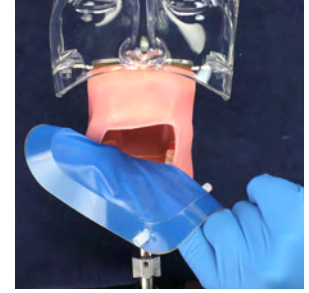

**6.** Pull on the stick for easy access to mouth for suctioning or inserting radiograph film.

**Autoclave sticks and save for re-use.**

**Sterilization Procedure:** Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.

## Package Contents:

Item No.	Dams Qty.	Sticks Qty.
I-70BM	20 dams	5 sticks
I-70BM-100	100 dams	20 sticks



**LATEX!** This product contains latex rubber, which may cause allergic reactions.

The Aseptico HandiDam is a single-use product. Discard after one use.

Protect from light  
Store in cool environment  
(Room temperature or cooler)

**HandiDam**® is manufactured exclusively by Aseptico. It saves time, aids in infection control, and preserves patient goodwill.



Easy to use, convenient,  
and saves time.



**LATEX!** This product contains latex rubber, which may cause allergic reactions.

The Aseptico HandiDam is a single-use product. Discard after one use.

Protect from light  
Store in cool environment  
(Room temperature or cooler)

**HandiDam**® is manufactured exclusively by Aseptico. It saves time, aids in infection control, and preserves patient goodwill.



Easy to use, convenient,  
and saves time.